

INSTRUCTION FOR USE

FOAM DRESSING

Product name: Foam dressing

Type&Size

4cm×5cm	5cm×5cm	5cm×12.5cm
5cm×6.2cm	5cm×7.5cm	5.7cm×8.2cm
6cm×7cm	7.5cm×7.5cm	8cm×8cm
8cm×8.4cm	8cm×15cm	8.6cm×7.7cm
9cm×9cm	9.5cm×8.5cm	10cm×10cm
10cm×12cm	10cm×15cm	10cm×20cm
10cm×25cm	10cm×30cm	10cm×35cm
10.3cm×10.3cm	10.5cm×13.5cm	12.5cm×12.5cm
12.9cm×12.9cm	13.1cm×15.2cm	15cm×8.5cm
15cm×15cm	15cm×20cm	15.4cm×15.4cm
16.8cm×17.1cm	17.1cm×17.9cm	17.5cm×17.5cm
17.2cm×17.5cm	18cm×18cm	20cm×20cm
21cm×21cm	21.6cm×23cm	22.5cm×22.5cm
23cm×23.2cm	25.2cm×25cm	25cm×25cm
25cm×30cm	30cm×30cm	

Product Structure and Composition

It is composed of adhesive polyurethane film, polyurethane foam, non-woven fabric, silicone gel, polyurethane film and release paper.

Range of Application

It is used to cover non-chronic wound surface and absorb wound exudate.

Method of Use

1. Clean and disinfect the wound and the surrounding skin.
2. Choose proper size dressing and peel off one side release paper.
3. Stick it on the wound and remove the other side release paper.
4. Smooths folds and fits wounds.

Replacement Frequency

In the early stages of wound treatment, always check the condition of the foam dressing. The dressing can be applied to the wound for up to 7 days, and can be replaced depending on the condition of the wound and the skin surrounding the wound, if any trace of effusion is seen up to 1.5 cm from the

edge of the dressing. The maximum cumulative use time of the product shall be no more than 30 days.

Contraindications

1. Use with caution if the skin is allergic to silicone glue.
2. Do not use on third-degree burns, severe infections and tumor wounds.







Precautions

1. Do not use if the package is open or damaged.
2. This product is disposable sterile products and cannot be reused.
3. Use within the product validity period.
4. The products shall be disposed of as medical waste after use to avoid pollution.
5. In case of skin redness or allergy, please stop using immediately.

Storage

It is recommended to store in a cool, dry, ventilated, non-corrosive gas environment, away from fire sources and combustible materials.

Symbols On Labeling Recautions

	Method of sterilization: Ethylene Oxide
	Do not reuse
	Do not use the damaged package
	See Instructions For Use
LOT	Batch number
	Date of Manufacture
	Use by-year and month

Shelf Life : 3 years

Manufacture Date : Refer to inner box

Manufacturer

SLK Medical Material (Tianchang) Co., Ltd.
No. 33, Zhenglong Road, Qinlan Town
Industrial Park, Tianchang City, Anhui
Province 239300, P.R. China

Tel: 0512-67339692