


Silicone Foam Dressing

Silicone Foam Dressing With Border

Size: 10cm x 20cm

Color: Pink, Skin


Tailored design for covering Cesarean section incision sites

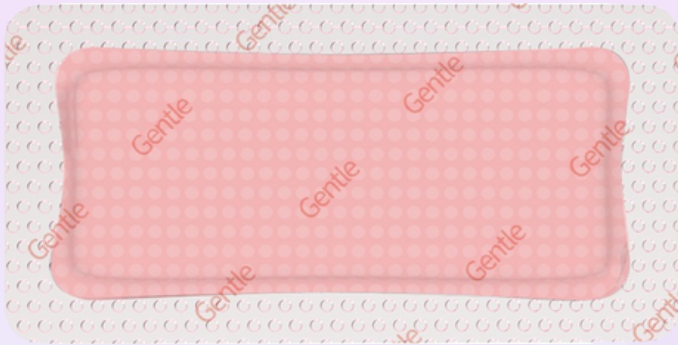

Cesarean Care Ultrathin have several potential benefits when used for the management of cesarean section incisions. These benefits can contribute to improved wound healing and patient comfort during the postoperative recovery period. Here are some of the advantages of using silicone foam dressings for C-section incisions

Advantages

- Reduce Surgical Site Infections
- Reducing the Darkness of Scars
- Reduced Dressing Changes
- Enhanced faster Healing
- Can Lift and re-adhere
- Water proof
- Maintains a Moist Environment
- Protection Against Infection
- Comfort and Pain Reduction
- Thin and Low Profile edge
- Minimal Trauma During Dressing Changes

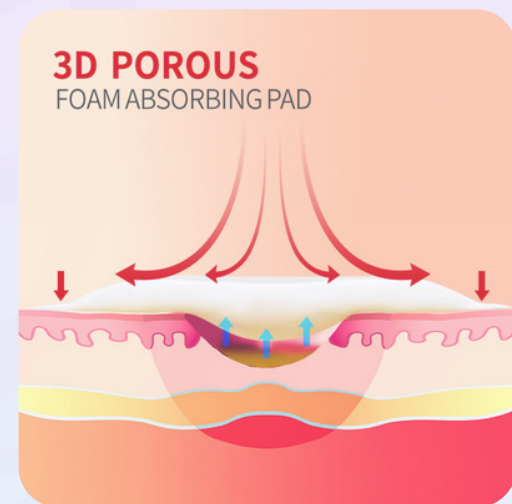

pH Value: 5.0~9.0

Appearance

The dressings is clean, uniform thickness and color, free from damage and foreign matter, silicone film is smooth and no damage, the hole is well distributed, the core is in the middle of the dressing, the colorful backing is smooth.

Silicone residue:: It should be no obvious residual gel on the release film.

Silicone adhesiveness: It should be able to adhesive up and keep it for more than 30 seconds.



Imported and supplied by

amigo care dhaka

Trusted Advance Wound Care Solution

We deliver prestigious certified products



Silicone Foam Dressing

Silicone Foam Dressing With Border

Size: 10cm x 20cm

Color: Pink, Skin

Apply the Silicone Care Ultrathin

- Cleanse the wound using a wound cleaning solution or sterile saline solution, as per healthcare provider's instructions.
- Choose a silicone foam dressing that is large enough to cover the entire incision area, with some overlap onto healthy skin for better adhesion.
- Carefully place the silicone foam dressing over the cesarean section incision, ensuring that it covers the entire incision and adheres well to the surrounding healthy skin.
- Press down gently to secure the adhesive border to the surrounding skin.
- If the silicone foam dressing includes adhesive strips for securing the edges, apply these strips over the dressing's edges to prevent them from lifting.

Contraindications

Do not use Cesarean Care Ultrathin on patients with known sensitivity to silver or any other contents of the dressing.

Warnings

- Cesarean Care Ultrathin is single use. Do not reuse.
- Do not use it if the pouch is damaged or opened prior to use.
- Do not suggest using this product on dry wound.
- Do not use Cesarean Care Ultrathin foam dressing together with oxidant, such as hypochlorite solutions or hydrogen peroxide.

Wear time: Up to 7 days

