

Silicone Foam Dressing

Silicone Foam Dressing With Border

Size: 7.5cm x 7.5cm, 10cm x 10cm, 12cm x 12cm

Color: Pink, Skin

Product Structure & Composition

It is composed of Adhesive Polyurethane film, Polyurethane foam, Silicone gel and Release paper.



Waterproof PU film backing

- Water proof , anti-bacteria
- Prevent external friction
- Managing the Microenvironment



Foam absorbent layer

- Offers additional comfort & cushioning protection
- Great fluid handling capacity
- Maintain a wet healing environment



Foam edge sealing technology

- Tight fit, lock exudate without leakage
- Reduce external friction.



Silicone wound contact layer

- Provides gentle care for fragile & sensitive skin
- Gentle self-adhesion



Imported and supplied by

amigo care dhaka
Trusted Advance Wound Care Solution

We deliver prestigious certified products




Silicone Foam Dressing

Silicone Foam Dressing With Border

Size: 7.5cm x 7.5cm, 10cm x 10cm, 12cm x 12cm

Color: Pink, Skin

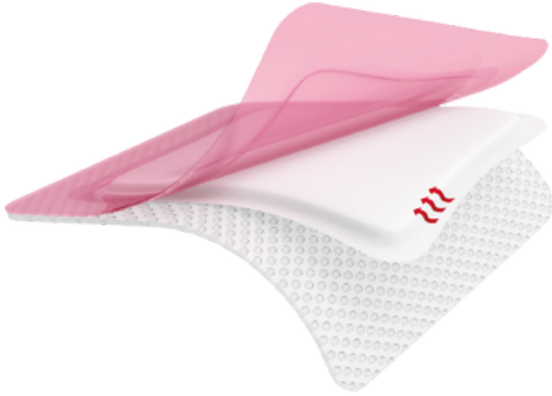

PRODUCT FEATURES

- Reduce Surgical Site Infections
- Reducing the Darkness of Scars
- Reduced Dressing Changes
- Enhanced faster Healing
- Can Lift and re-adhere
- Water proof
- Maintains a Moist Environment
- Protection Against Infection
- Comfort and Pain Reduction
- Thin and Low Profile edge
- Minimal Trauma During Dressing Changes

How to Apply

Assess the Wound:

- Note any signs of infection, such as redness, warmth, swelling, or increased pain.
- Examine the wound to assess its size, depth, & the amount of exudate (fluid drainage)

Cleanse the Wound

- Cleanse the wound using a prescribed wound cleaning solution or sterile saline solution, as per healthcare provider's instructions.
- Gently irrigate or cleanse the wound to remove debris and excess exudate.
- Pat the wound dry with sterile gauze or a clean, lint-free cloth, if needed.

Apply the Silicone Care Ultrathin

- Open the packaging of the silicone foam dressing while wearing clean gloves.
- Carefully remove the dressing from its packaging, taking care not to touch the adhesive side with your fingers.
- Place the dressing directly over the wound, ensuring it covers the entire wound bed.
- Press down gently to secure the adhesive border to the surrounding skin.

Wear time: Up to 7 days

Age Limits: No Age limits

